


SAVE THESE INSTRUCTIONS FOR IMPORTANT OPERATING, AND PARTS INFORMATION

Easy Roller™ Non-Ambulatory Roller HM1045

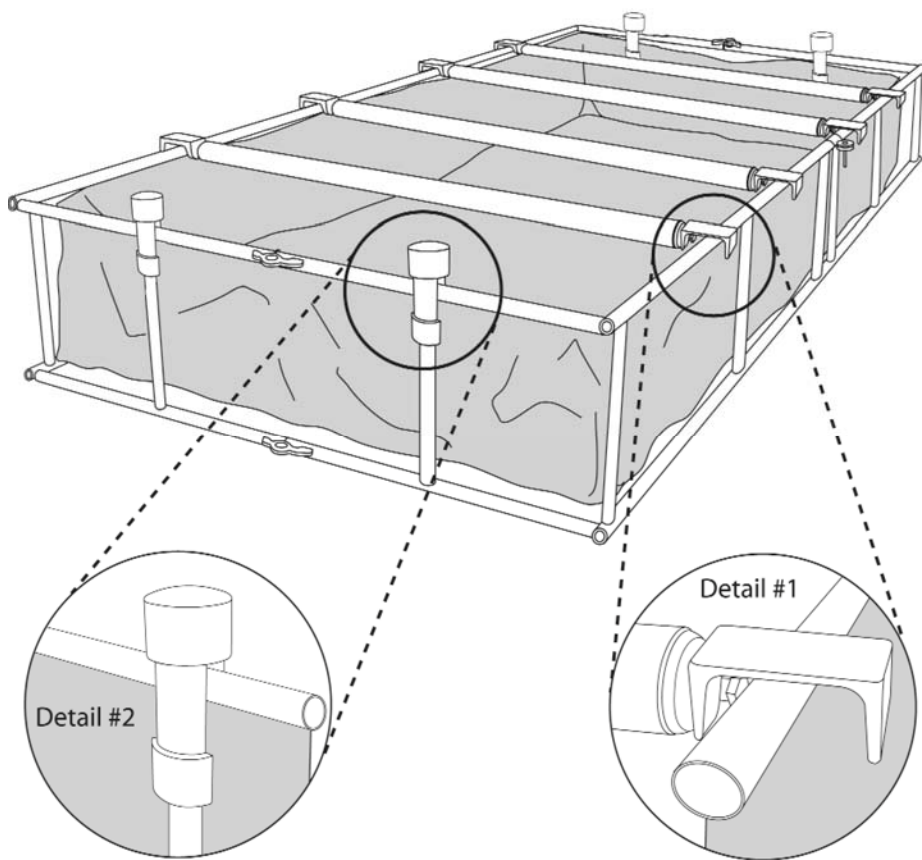

Assembly

- A. The collection pool must be positioned on a level surface to prevent unwanted backboard movement on the roller system.
- B. Pull all four rollers from bag.
- C. If shower is erected in the pool, be certain that the shower is off and the enclosure sides are rolled up and out of the way.
- D. Place each roller across the width of the pool on the frame's edge, spaced every 18-19". (See Detail #1)
- E. Apply backboard stops to the ends of the pool so that they straddle the pool edge and will stop unwanted backboard movement. (See Detail #2)
- F. Roll patient through system. Use caution when logrolling patient to prevent spinal injury. Handheld sprayer should be used instead of shower. Patient must never be left unattended.
- G. Rollers are durable and reusable and should be cleaned in the same manner as the other decontamination equipment. Dry rollers thoroughly prior to storage.

Parts List Replacement parts are available, contact DQE to order.

| PART NAME | QTY | PART NUMBER |
|---------------------------|-----|-------------|
| 1. Stainless Steel Roller | 4 | HMPT10452 |
| 2. PVC Backboard Stop | 4 | CT10454 |